

Alan-Glyde Suture



Easy Retrieval Pack (directions for use)

1 Tear open the foil at the notch in the direction shown



2 Pull the torn portion down as shown till the needle is visible



3 Hold the needle with the needle holder and pull the suture out



ALAN-GLYDE

- Sterile, synthetic surgical suture
- Absorbable
- Multifilament
- Braided and coated polyglycolic acid, composed of homopolymers of Glycolide (100%)
- Violet coloured with D & C Violet No.2
- Coated with unique combination of polycaprolactone and calcium stearate
- Loss of tensile strength : 75% at 2 weeks, 50% at 3 weeks, 90% loss of tensile strength after 4 weeks
- Complete absorption after 10 weeks (70 days)
- Needle made of stainless steel, silicone coated. Eases the passage without damaging the tissue

Comments:

Class III medical device, single use product


Shelf life: 5 years from production date

Should be stored in a clean and dry place, out of direct sunlight and below a temperature of 30°C

Box size: 62x130x62 mm

Sizes

Reference	Needle profile	Needle description	Needle length (mm)	Size	Thread length (cm)
SUTPG3	3/8	Reverse cutting	19	3/0	70
SUTPG4	3/8	Reverse cutting	19	4/0	70
SUTPG5	1/2	Reverse cutting	16	5/0	70

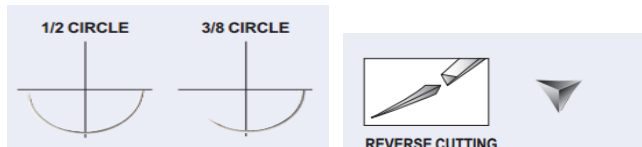

Packaging

Number of piece/box	Number of boxes/export box
12	216

We are ISO 13485 certified and our products are CE marked



ALAN & CO

Avenue André Ernst 17 – 4800 Verviers – Belgium

Tel : +32 87 31 10 59 – Fax : +32 87 31 61 69

www.alan.be – info@alan.be

version 08/2018