

Alan Lact Fast suture



Easy Retrieval Pack (directions for use)

1 Tear open the foil at the notch in the direction shown



2 Pull the torn portion down as shown till the needle is visible



3 Hold the needle with the needle holder and pull the suture out



ALAN LACT FAST	<ul style="list-style-type: none"> • Sterile surgical suture • Absorbable • Multifilament • Composed of : 90 % Glycolide, 10 % Lactide : Polyglactin 910 • Violet coloured with D & C Violet No.2 • Coated with combination of Polyglactin 370 and calcium stearate • Total loss of tensile strength: 10 to 14 days • Complete absorption after +/- 42 days • Needle made of stainless steel, silicone coated. Eases the passage without damaging the tissue
-----------------------	---

Comments:

Class III medical device, single use product


Shelf life: 5 years from production date

Should be stored in a clean and dry place, out of direct sunlight and below a temperature of 30°C

Box size: 62x130x62 mm

Sizes

Reference	Needle profile	Needle description	Needle length (mm)	Size	Thread length (cm)
SUTLF3	3/8	Reverse cutting	19	3/0	70
SUTLF4	3/8	Reverse cutting	19	4/0	70
SUTLF5	1/2	Reverse cutting	16	5/0	70

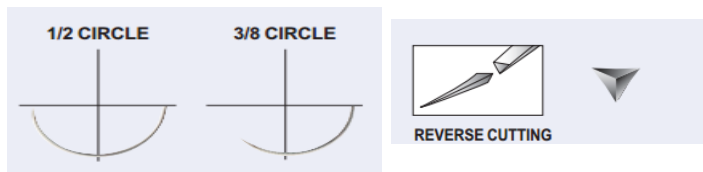

Packaging

Number of pieces/box	Number of boxes/export box
12	216

We are ISO 13485 certified and our products are CE marked

